

Parameters & Features



2-channel ECG



Heart Rate



5-Day Wear



Phone Symptom Indicator



Water Resistant



No Cable
Connection Required



One-Time Use
Infection Control Concerns

Patient FAQs

Do I need to keep the mobile phone near me?

Yes, it is imperative to keep the phone provided within 10 feet of the patch and charge it at least every 12 hours to ensure continuous streaming of data. A fast-flashing green light will be seen if the patch is away from the phone.

Do I have to press the button on the patch?

Firmly press the patch ON button just once at the start of your monitoring only.

Can I take a shower wearing my patch?

Yes, keep showers short with your back to the flow of water while showering. Do not use creams, shampoo, lotions, or similar products near the patch. After your shower, gently pat dry with a towel and minimize activity until the patch is dry.

What if my patch accidentally gets wet?

If the patch accidentally gets wet, gently pat dry with a towel and minimize activity until the patch is fully dry.



866-951-2142
713-821-3200
artellainc.com



2-Lead Wireless Disposable Device

Patient Information & Instructions




Start Monitoring




1 Perform Skin Preparation for Better Connectivity

- If required, remove excess hair from upper left chest
- Clean the area with non-moisturizing soap and water
- Rinse the area making sure you remove all soap residue
- Ensure the area is completely dry

Note: Avoid using wipes or isopropyl alcohol to clean the skin as alcohol dries the skin and increases the possibility of irritation which can reduce the electrical signal to the Patch.

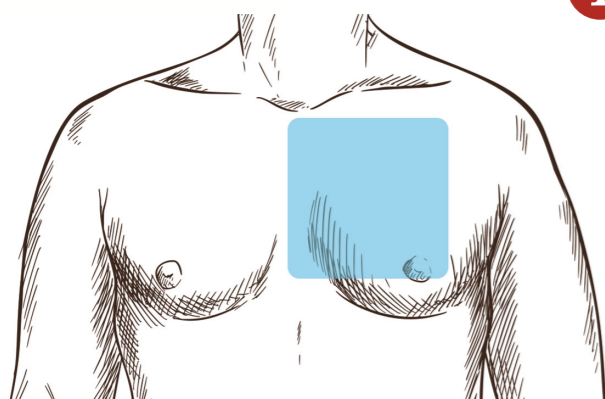

Right Upper Electrode Left Upper Electrode

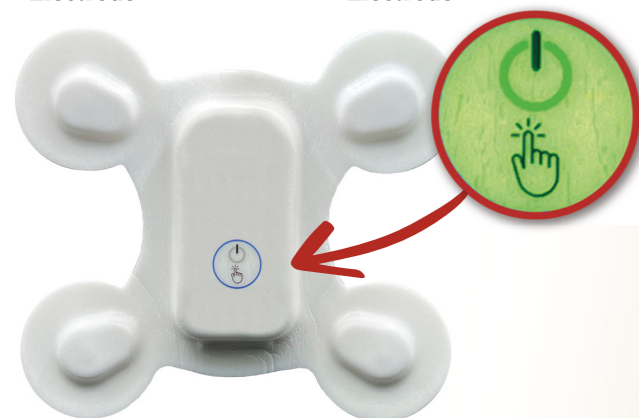

2 Turn the Patch ON

Press the patch ON button once. A red light will flash followed by a fast-flashing **green light**. Once connected, the flashing **green light** will slow down.

Patch Light and Status



Fast Flash:
Patch is trying to connect to App



Slow Flash:
Patch is connected to App



Slow Flash:
Low battery indication




Fast Flash, Then Off:
"Turned off"

Right Lower Electrode Left Lower Electrode

3

Apply Patch

- Gently peel off the backing film
- Place the patch on the upper left chest, below the collar bone and to the left of the sternum
- Press the entire patch, especially the four circular corners, for 2 minutes
- Limit activity for 1 hour post placement to ensure good skin adherence

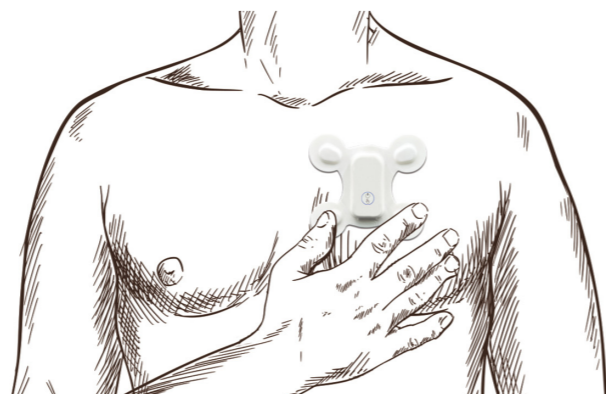

Report Symptoms During Monitoring

It is important for your physician to know if you have any symptoms while wearing the device so they can correlate them with your readings.

4 To report symptoms while monitoring:

- Open Artella MCT app on the phone (red circle shown) and press on the appropriate picture to reflect your symptom felt
- You may select more than one symptom if required




Additional Monitoring

An additional patch may be assigned to initiate another monitoring session. Please call our Elite Team for assistance when hooking up a 2nd patch.

End Monitoring

- When monitoring is complete, please remove the patch slowly from the outer circles towards center of the patch and dispose of it.
- Use the provided adhesive remover if needed.
- Return all components (charger and phone) with the return envelope provided.

Troubleshooting

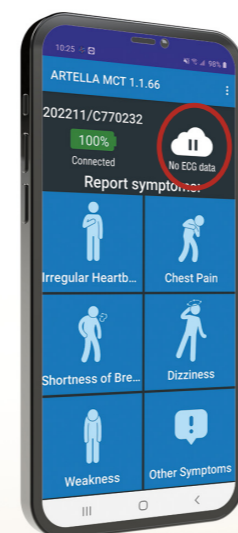

If a Patch corner begins to loosen or lose contact with the skin, the displayed alert (red circle) may be shown on the phone. Press all corners firmly with your fingers to reconnect.

Avoid use of the Patch near any electromagnetic anti-theft devices, metal detectors, or MRI machines.

DO NOT USE if you have a known allergic reaction to adhesives or electrodes

DO NOT USE if you have inflamed, irritated or broken skin in the patch placement area

DO NOT REUSE THE PATCH. IT IS FOR SINGLE USE ONLY. IT IS NOT A LIFE SAVING DEVICE.

For assistance, please call
866-951-2142
713-821-3200